



400 Apgar Drive, Suite I, Somerset, NJ 08873  
P: 800 851 0025 F: 731 356 0070

[sales@software.com](mailto:sales@software.com)  
[orders@ec2software.com](mailto:orders@ec2software.com)  
[support@ec2@software.com](mailto:support@ec2@software.com)

### **Ec<sup>2</sup> Software Solutions guidelines for Citrix Server Environment:**

1. Dedicated or shared server stores all programs and data. Also is installed as a workstation so clients can connect to server through terminal or Citrix client and run NMIS\Biodose.
2. NMIS\Biodose needs to be installed as a server and workstation on the Citrix server.
3. In the case of multiple installs of NMIS\Biodose on one Citrix Server, Folder structure needs to be different for each install. (ie: nmisxxx for site a and nmisxxx1 for site b).
4. Once install is complete you will have an icon on the desktop for each site. Client computers should now log in to server via terminal environment\Citrix environment.
5. NMIS\Biodose licenses will now be based on sessions. Each session is a license and should be purchased according to how many workstations will be connecting to the server for NMIS \Biodose purposes.
6. Local client printers will need to be shared and drivers installed on the server.
7. Memory is the most important factor when considering installing on a Citrix Server. The number of instances of NMIS\Biodose, any other programs and number of clients connecting will all be a factor.
8. If you are migrating data, this will happen after all the instances are installed and configured. A current backup of each instance will be needed to perform the restore.

Ec<sup>2</sup> Software Solutions Support

