


NumaBridge™

Legacy System Isolator

Remove Security Risks From Your Network Without Sacrificing Functionally

NumaBridge Features:

- Provides security by isolating the acquisition systems and workstations running Windows XP / 2000 from the hospital network (2 network interface cards are required)
- NumaBridge supports Windows 7/10 updates, anti-virus and other current security measures mandated by IT departments.
- NumaBridge works with NumaLink to translate non-DICOM files to DICOM or correct DICOM incompatibilities.
- NumaBridge is compatible with any DICOM compliant system and NumaLink

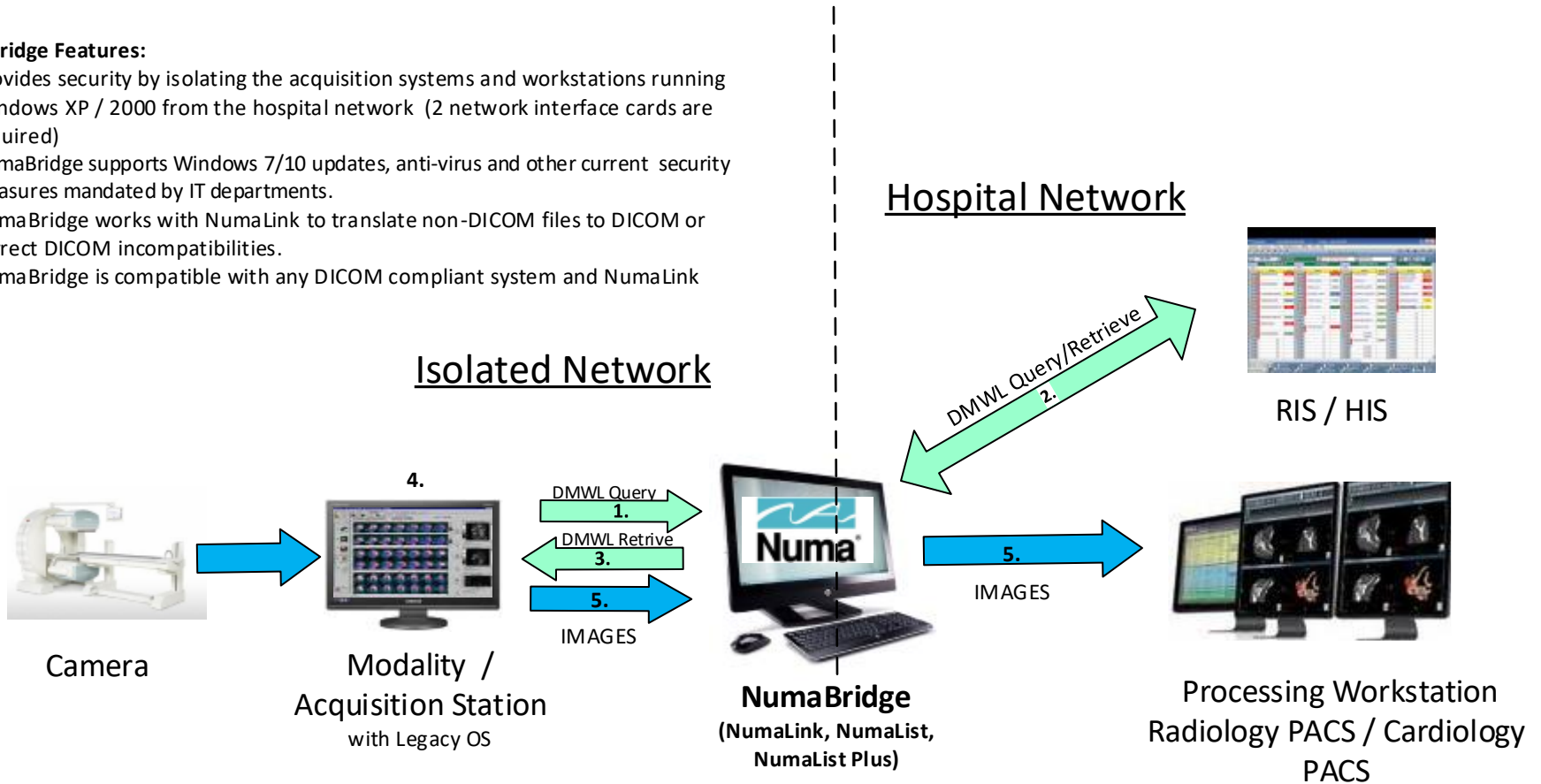

NumaBridge Functionality:

1. The user performs a DICOM Worklist Query for a list of scheduled patients from the modality for acquisition station.
2. NumaBridge passes the query from the modality on the isolated network to the HIS/RIS DICOM Modality Worklist (DMWL) provider on hospital network.
3. The results of the Worklist query are returned to the modality on the isolated network.
4. The user selects the current patient and acquires the studies normally.
5. When the studies are complete they may be manually or automatically exported to NumaBridge.
6. NumaBridge automatically forwards the DICOM studies to the destination system on the hospital network.